

# Roll and bag an Auto Logic mattress



1 Hold down power button to switch off pump



2 Disconnect power cable from socket




3 Remove cable from cable management system



4 Wrap cable around the bottom of the pump




5 Release CPR by squeezing the sides

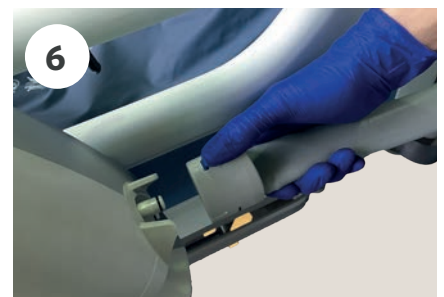

6 Disconnect tubeset from pump



7 Place tubeset on mattress and roll from the foot end towards the head end (roll it once to expel majority of the air, then retry rolling it to achieve a tighter roll)

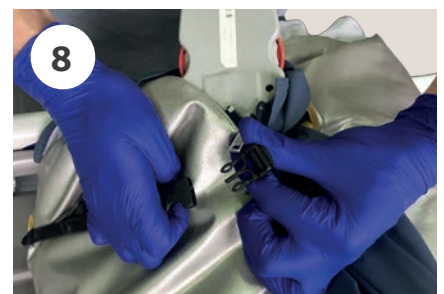

8 Use buckles to secure the mattress



9 Bag mattress and pump



10 Place the pump on top of the mattress in your designated collection area

For more information call  
Arjo Rental: 1800 00 7368  
or email:  
[customerservice-au@arjo.com](mailto:customerservice-au@arjo.com)

