


# Maxi Sky 2

## Transfer from seated position




**1**




Place the sling around the patient. For sling with head support make sure it extends to top of patient's head.


**2**

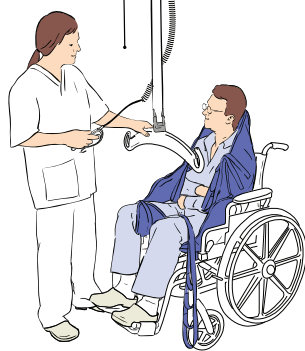


Approach the lift. Place spreader bar at shoulder level and close enough to attach shoulder straps.

**3**

 Clip sling


 Loop sling

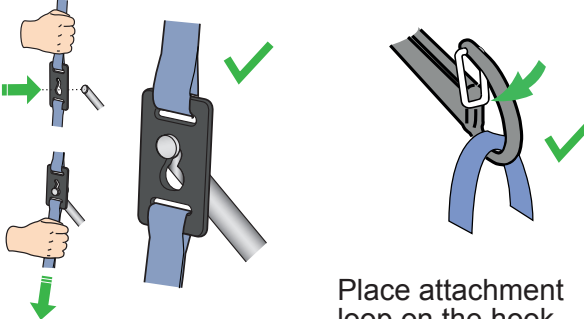

Attach and secure the clip at the lug before patient is lifted.


Place attachment loop on the hook. Make sure that the safety latch is closed.

**4**

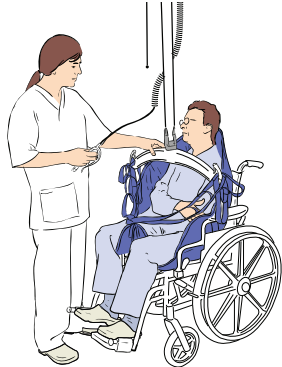


Connect the sling loops or clips to the spreader bar.

**5**

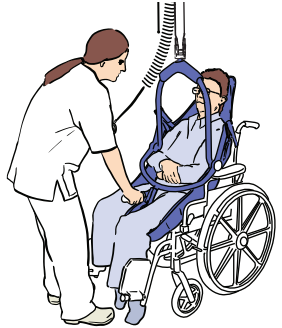


For the DPS spreader bar and clip sling, tilt the spreader bar to attach the leg clips.

**6**

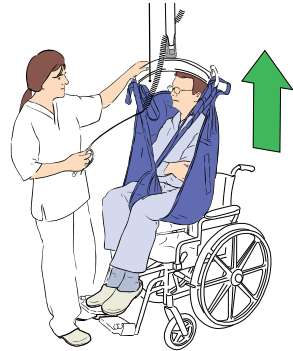


Raise the patient.

**7**

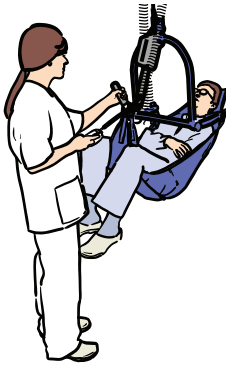

Move the patient.

**8**

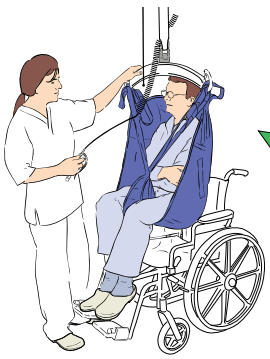

Lower until no tension. Detach loops or clips.


## Transfer from supine position

**1**



Place and centre the sling under the patient. For sling with head support make sure it extends to top of patient's head.


**2**

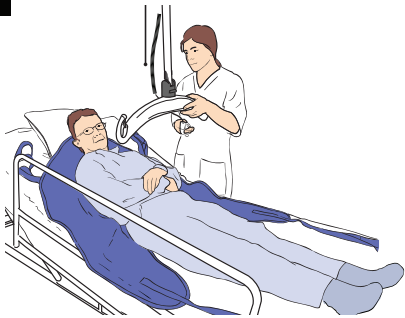

Position lift over the patient.


**3**

**! Clip sling**

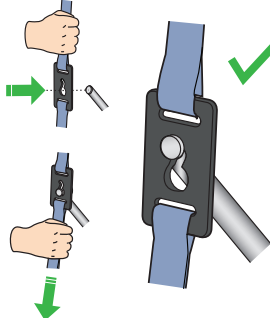

Attach and secure the clip at the lug before patient is lifted.

**! Loop sling**

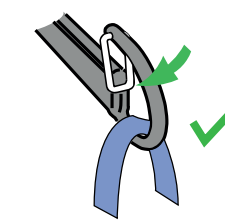


Place attachment loop on the hook. Make sure that the safety latch is closed.

**4**




Connect the sling loops or clips to the spreader bar. Attach shoulder straps then leg straps.

**5**

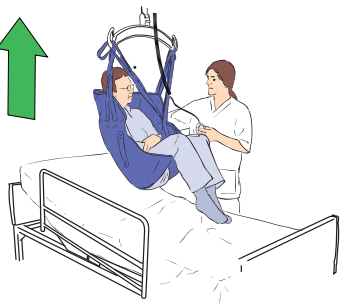


Raise patient.

**6**

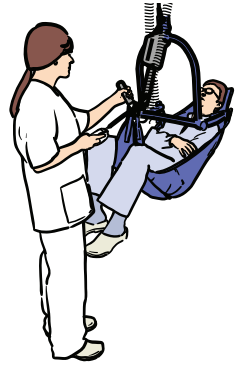


Move the patient.

**7**

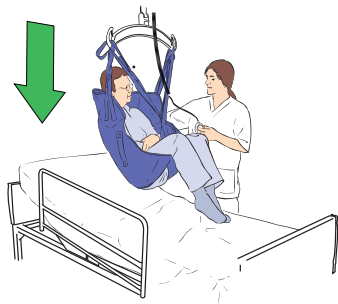






Lower patient onto new location.

**8**




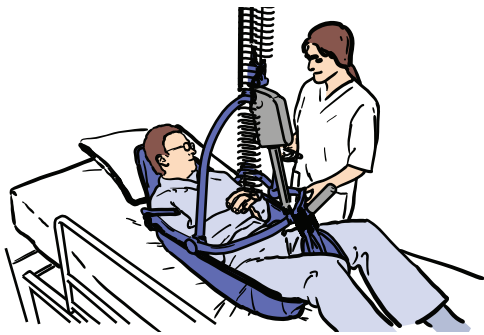


When the patient's body weight is fully supported, detach the loops or clips.

## Repositioning - move up in bed

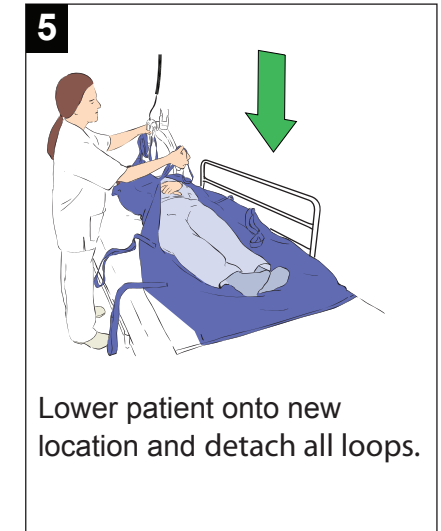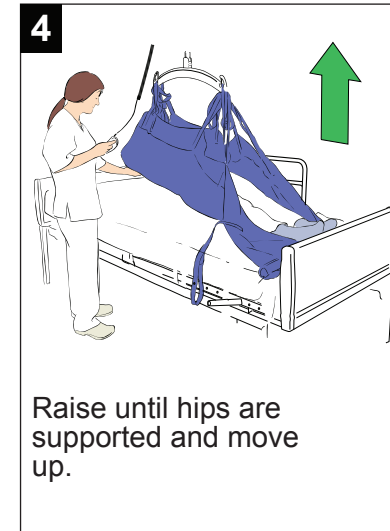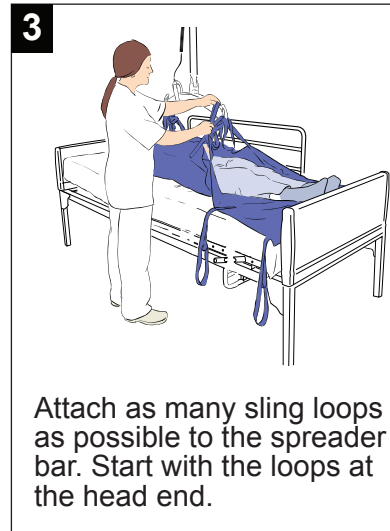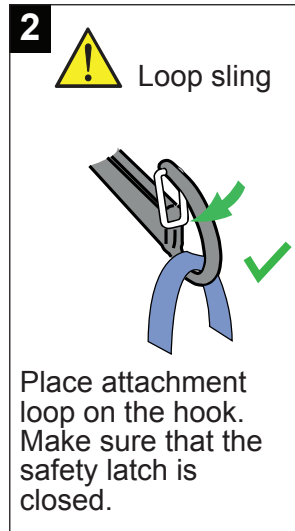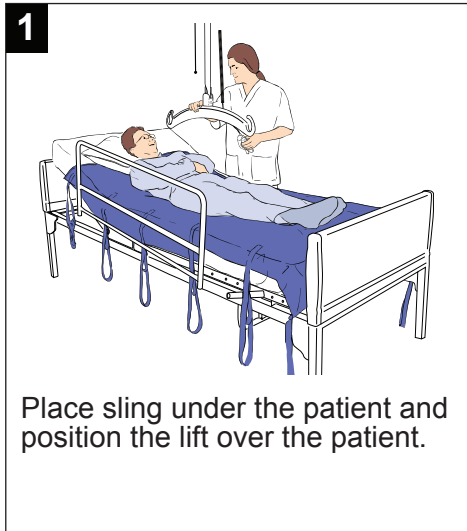

## Repositioning - bed turning

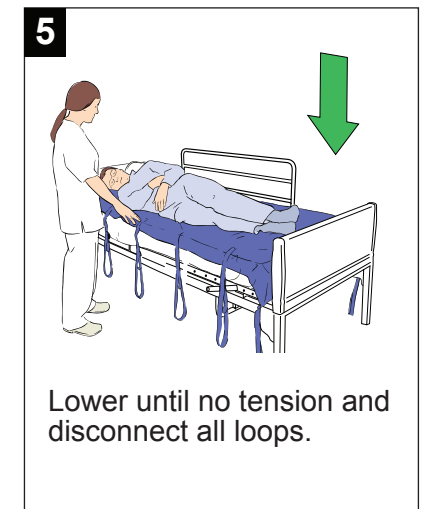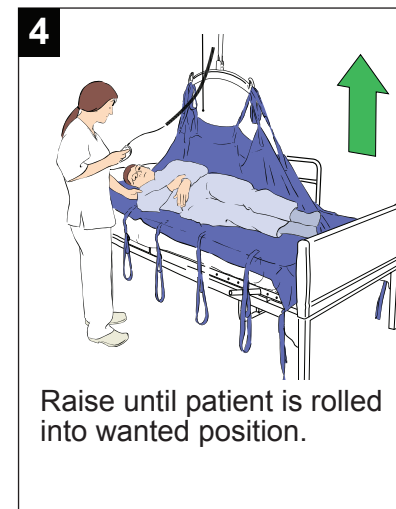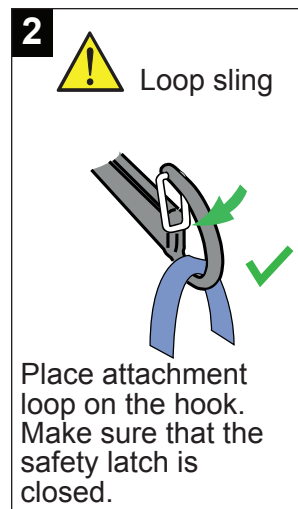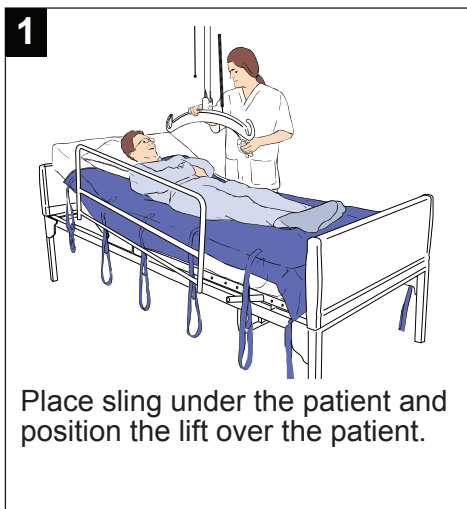

## Transfer patient in supine position

