

Maxi Sky 2



WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the instructions for use and material safety data sheet of the disinfectant.

| Allowed disinfectant agents | Quaternary Ammonium ≤ 0.21% (2,100 ppm) | Isopropyl Alcohol ≤ 70% | Phenolic ≤ 1.56% (15,600 ppm) | Bleach ≤ 1% (10,000 ppm) | Peracetic Acid ≤ 0.25% (2,500 ppm) | Hydrogen Peroxide ≤ 1.5% |
|---|---|----------------------------|-------------------------------------|--------------------------------|--|-----------------------------|
| Basic Ceiling Lifter model and Maxi Sky 2 Ceiling Lifter model (woven nylon strap) | X | X | X | | | |
| Maxi Sky 2 Infection Control model (non-porous strap) | X | X | X | X | X | X |
| Other Accessories * and ** | X | X | X | X | X | X |
| Medium 2-hook spreader bar (700-05461-BOX) & 4-hook spreader bar (700-05443) | X | X | X | | | |
| Stretcher Frame (700-19522) & Morgue stretchers (700.05380, 700.05385, 700-05382 & 700-05387) | X | X | X | | | |
| Manual Open DPS (700-19340-BOX) | X | X | X | X | | |
| Medium 2-hook spreader bar (700-19415) | X | X | X | X*** | X*** | X*** |
| Small 2-hook spreader bar (700-19420) & Medium 2-hook IC spreader bar (700-19470) | X | X | X | X | X | X |
| 4-hook spreader bar (700-19430) & Stretcher Frame (700-15695) | X | X | X | X | X | X |
| Manual Flat DPS (700-19475 & 700-19200) | X | X | X | X | X | X |
| Manual Open DPS (700-19480) | X | X | X | X | X | X |
| Medium PDPS (700-19350) & Large PDPS (700-19355) | X | X | X | X | X | X |

* Metal portion of the WCS connector may corrode after repeated exposure to bleach & hydrogen peroxide.

** Handset, WCS and PDPS cables may become slightly grimy after repeated contact with bleach or hydrogen peroxide.

*** The cover for this particular model may be vinyl padding or plastic. If vinyl, do not use these disinfecting agents.

Clean and disinfect before new patients.

Accessories needed:

- Protective glasses and gloves
- Disposable cloth

Clean

1. Use a cloth soaked with water to remove excessive residues from the lift and its accessories from top to bottom.
2. Use a damp cloth with warm water mixed with a cleaning fluid.
3. Scrub the equipment from top to bottom to remove any deposits.
4. Use a clean damp cloth with clear water to wipe off all spots of cleaning fluids.
5. Then wipe clean with a dry cloth.

Areas listed are most likely to contain dirt and microbes, pay attention to:

- Edge of hand control
- Back and handle of hand control
- Joint between spreader bar and strap
- Frame and cover of spreader bar
- Emergency cord
- Strap

Disinfect

6. Wipe the Maxi Sky 2 and accessories using a solution containing one of the compatible disinfectants.
7. Disinfection methods should comply with the local or national guidelines, depending on the Healthcare Facility or country of use. If uncertain, consult local Infection Control specialist.

