

Citadel Patient Therapy and Bed Frame System



Clean and disinfect weekly, and before a new patient.

Daily cleaning Citadel System

Daily care and cleaning consists of wiping down all surfaces and side rails.

1. Follow care and cleaning instructions for the specific patient support surface in use.
2. Remove patient from the bed prior to cleaning, if possible.
3. Unplug the Citadel System from the wall outlet.
4. Inspect the power cord for signs of wear or damage. The Citadel System should not be operated with a worn or damaged power cable.
5. Wipe the surfaces with a cloth dipped in warm soapy water or approved hospital disinfectant.
6. Rinse with a cloth moistened with clean water.
7. Allow all components to dry completely before returning to use.
8. Inspect all parts for damage prior to use.
9. Plug in the Citadel System to the wall outlet and adjust settings.

Cleaning and disinfecting Citadel System between patients

Cleaning

1. Remove the mattress and all accessories from the bed.
2. Remove the head and foot boards and mattress platform sheets before cleaning.
3. Clean all surfaces with a disposable cloth moistened in warm water and a neutral detergent.
4. Start at the upper sections and work along all horizontal surfaces. Work towards the lower sections of the bed and clean the castors last. Pay attention to areas that may trap dust or dirt
5. Wipe with a new disposable cloth moistened with clean water and then dry with disposable paper towels.
6. Allow cleaned parts to dry.

Disinfecting

7. After cleaning the bed, wipe all surfaces with sodium dichloroisocyanurate (NaDCC) at a concentration of 1,000 parts per million (0.1%) of available chlorine.

In case of pooling body fluids, e.g. blood, the concentration of NaDCC should be increased to 10,000 parts per million (1%) of available chlorine.
8. Wipe with a new disposable cloth moistened with clean water and dry with disposable paper towels.
9. Place the mattress back on the bed.

