


Auto Logic

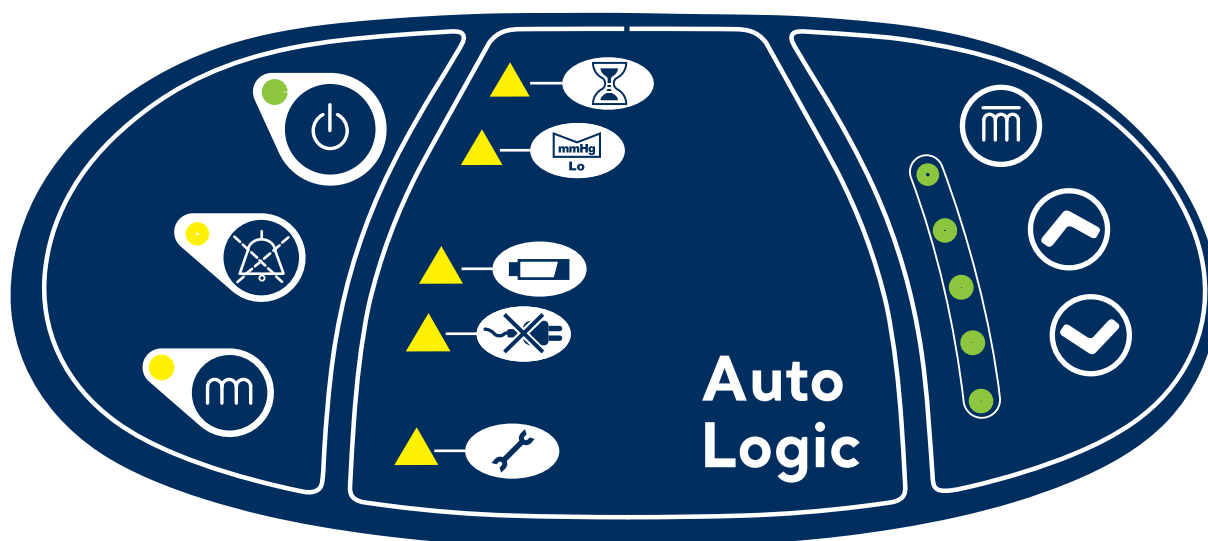

Run/Standby button

Press to activate the pump; the green indicators on the control panel and the front of the pump illuminates.

Press and hold 3 seconds for Standby.

Green indicator blinks twice every 2 seconds to indicate Standby.



Mute button

Press to mute the audible alarm. The yellow indicator is illuminated when the audible alarm is muted.



Reactive (CLP) mode

Press for Reactive (Constant Lower Pressure), non-alternating mode; the yellow indicator illuminates.

Press again for Active (alternating) mode (default); yellow indicator is switched off.



Autofirm mode

Press and hold 2 seconds for Autofirm mode.

Creates a temporary firm surface to allow nursing procedures. Reactive (CLP) and Comfort Control indicators are illuminated.

After 15 minutes the pump returns to the previous setting - Comfort indicators flash + series of tones. To extend 5 minutes: press either comfort button.

Maximum duration is 30 minutes.



Comfort control

Press buttons to adjust mattress or seat cell pressure for patient comfort. The pressure setting is shown by green indicator to the left.



Wait indicator

Illuminated until the mattress/seat has been fully inflated.



Low Pressure indicator

Illuminated, with audible alarm, when the pump detects low pressure in the mattress/seat.

Once normal pressure is reached the indicator is turned off.



Battery Low indicator

Illuminates 2 hours before battery failure.

The pump defaults to Reactive (CLP) mode 1 hour before battery failure, there is an audible alarm and the yellow indicator is still illuminated. (Battery pack optional)



Power Fail indicator

Illuminated, with audible alarm, when mains power failure is detected and no battery is available. The alarm continues until mains power is resumed or pump is switched off.

Disconnect the tubeset from the pump to put the mattress in transport mode.



Service indicator

Illuminates after a set number of running hours, indicating the pump is ready for service. The pump continues to function normally while the Service indicator is on.