


Citadel Patient Therapy System

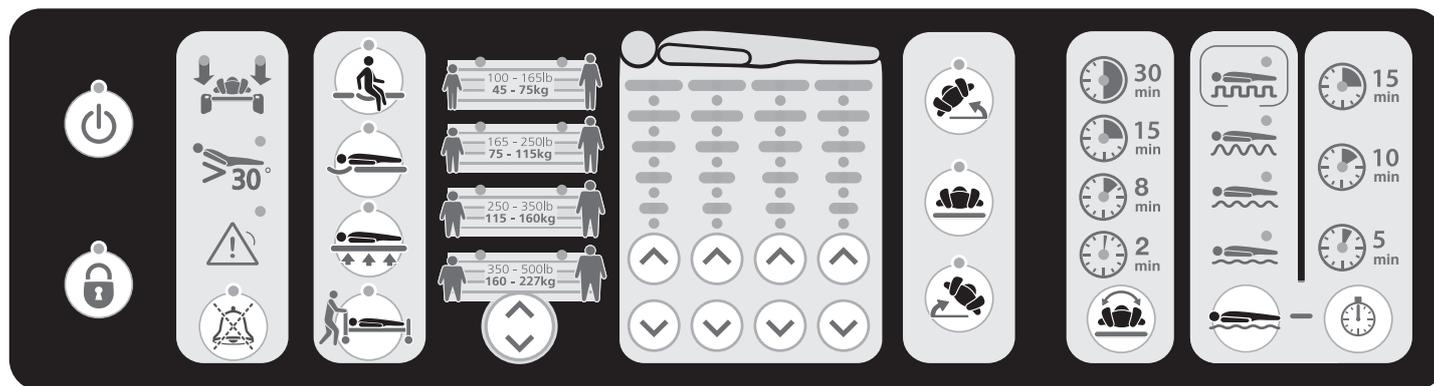

Power On / Off Button

Press and hold for two seconds to turn on or off.



Lock Out Button

Press and hold for two seconds to activate or deactivate lockout of all functions.



Side Rail Down Indicator

Indicates when the left or right side rails are down.



Fowler Angle Indicator

Indicates when fowler angle is greater than 30°.



Alarm Indicator

Indicates when an alarm condition exists.



Alarm Silence / Clear Button

Press to clear an alarm indicator or to acknowledge an alarm.



Seat Deflate Button

Press to activate or deactivate seat deflate.



Head Deflate Button

Press to activate or deactivate head deflate.



Firm Mattress Button

Press to activate or deactivate firm mattress function.



Patient Transport Button

Press to activate or deactivate patient transport.

Turning



Patient Turn Right Button (nurse assist)
Press to rotate the patient 20° to their right.



Patient Center Button
Press to return from a turned position to a flat.



Patient Turn Left Button (nurse assist)
Press to rotate the patient 20° to their left.



30 min

Continuous Patient Turn Hold Time Button (optional configuration)



15 min

Press to initiate continuous patient turn.



8 min



2 min




Air Pressure Adjustment Controls



Height / Weight Presets

Press button to select the patient body type and weight.

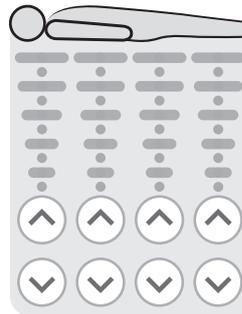


Pressure Zone Indicators

Press the up arrows to increase pressure.

Press the down arrows to decrease pressure.

Pulsation/Alternating Pressure Therapy (optional)

Alternating Pressure

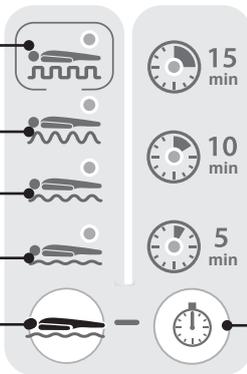

High Pulsation

Medium Pulsation

Medium Pulsation

Pulsation / AP Therapy Button

Pulsation / AP Cycle Time Button



Pulsation / AP Therapy Button
Press to adjust intensity of pulsation therapy.



Pulsation / AP Cycle Time Button
Press to select the pulsation cycle time.